

SCENAR COSMODIC VX735L

Professional Devices



SCENAR COSMODIC VX735L is a professional SCENAR COSMODIC series model. Increased reliability, first-class performance, high efficiency, and easy use. Intended for prevention and treatment of diseases and pathological conditions.

Innovations

Three technologies of action — **COSMODIC, SCENAR and SCENAR MAX** always allow you to effectively treat a wide range of diseases.

Minimal and maximal doses are implemented for the first time — now defining time necessary for treatment is much easier and more accurate.

New function of digital three-stage analysis of the treated area allows you to easily control treatment and quickly find areas which would best respond to action.

VX735 — is the first professional model incorporating combined SCENAR-COSMODIC technology. After 7 years of serious on-going improvements and updates “LET Medical” achieved peak performance for the VX735. Now the device is completely renewed and presented with two models SCENAR COSMODIC VX735Ag and SCENAR COSMODIC VX735L

Main features:

- parameters of the device tune automatically in to a particular person at switch on;
- microbioresonance technology provides higher sensitivity of the device and allows getting more accurate biological feedbacks of the body;
- stimulation during prolonged action is managed automatically;
- first-aid effect;
- three action modes: mode for treatment of diseases of nervous system and accompanying disorders, mode for treatment of pathological conditions of internal organs, and mode providing maximal action energy for treatment of locomotor disorders;
- timer showing time of action upon the particular area and total time of action during the session.
- LCD BW display;
- LED and sound indication;
- connecting remote electrode;

Indication

LCD display

Shows action modes, action energy level, state of the treated area, its activity during treatment and changes, dosing, time of action upon a particular area and total time of the session, and battery status.

LEDs

Allow you to monitor reaction of the body to action, switch on of dynamic adaptation mode and contact of electrodes with skin.

Action modes

“CS” — **COSMODIC** — main mode for treatment of diseases of nervous system and accompanying disorders.

“SC” — **SCENAR** — is used for treatment of various pathological conditions of internal organs.

“SM” — **SCENAR max** — provides maximal action energy for treatment of locomotor disorders.

Electrodes

The model has built-in electrodes on the front-end part of the device. The electrodes are made of special steel which has a pyramidal structure. An important peculiarity of such kind of steel is that chrome does not penetrate into skin due to electrolysis process occurring during action.

In addition, you can order electrodes installed on the under side of the device, which can be used for long action upon large areas.

Casing

SMART series casing made of shockproof ABS type plastic, 3.5 mm thick.

Waterproof keyboard.

Size

143 x 43 x 30 mm

Power supply

Two galvanic cells or rechargeable batteries, AA type, 1.5V.

Delivery set

- Device
- User's Guide
- Package



Models of the 735th series are very popular among practitioners. One of the key advantages is that you can select technology of treatment manually, which creates favourable conditions for more efficient professional work. This device is also a kind of trade-off for those who spent many years studying methods and protocols of treatment necessary for SCENARs of older generations. It is a good choice to “switch to” automated devices

Touch of Power Therapies
PO Box 239
Mapleton
Queensland 4560
Australia
www.touchofpower.com.au
ph 07 5445 7778
m 0407 136 794